UZAARM

Health Care Family Installation Manual

UZAARM-200

- EN** **Caution**
1. Read this manual completely and carefully before installation and follow the instructions herein.
 2. Keep this manual well for further reference.
 3. Use original parts only. Install with any other unapproved components may result in the falling off or malfunction of display unit and this wall mount ARM bracket.
 4. Check all the parts as shown on the manual before install. If parts are missed or damaged, please keep it out of use and return it to local retailer or original seller.
 5. After wall mount ARM being installed, please check the bolts and all fastening devices every 36 months. Dismantle the display unit immediately and keep it out of use if any of them get loose, rusty or lost. Return defective ARM & associated parts to original seller or local retailer for counter maintenance.
 6. This wall mount is intended for patient care use with the maximum weight indicated only, any overloading will lead to falling hazards.
 7. Please check the wall substance and thickness being suitable for the install of subject Long ARM. This wall mount ARM is applicable for vertical, even solid masonries, concrete or brick wall only !. For other application, please consult licensed professional to evaluate the feasibility.
 8. Do not disassemble or repair the product by yourself. Warranty will be void after disassembled!
 9. According to the illustrations, use cable binder or other similar damper to fix the cable in order to prevent from unintentional shear risk and electric shock.
 10. The manufacturer shall not be legally responsible for any equipment damage or personal injury caused by incorrect installation or operation other than that covered in this manual.
 11. The wall mount is designed for easy installation and removal. The manufacturer shall not be liable for damage to equipment or personnel injuries arising out of human factors or Acts of God, such as earthquake or typhoon, etc.
 12. It is recommended that the wall mount ARM/bracket and display unit to be installed by qualified personnel with using default attached bolts/accessories only.
 13. At least two persons are needed to install or remove the product so to avoid hazard of falling objects.
 14. Please carefully inspect the area where the wall mount ARM/bracket will be installed:
 - Avoid places that are subject to high temperatures, humidity, or contact with water.
 - Do not install the product near air conditioning vents or areas with excess dust and fumes.
 - Only install on vertical walls and avoid slanted surfaces.
 - Do not install in places subject to any shock or vibration.
 - Do not install in places subject to direct exposure to bright light, as it may cause eye fatigue when viewing the display panel.
 15. Maintain sufficient space around the display to ensure adequate ventilation
 - Please install and fix the display unit first before wire harness to be connected on it.
 - Reserve sufficient space between patient & anchor base of articulating arms for essential turn-around of Long ARMs.
 16. To ensure safe installation, first check the structure of the wall and select a secure mounting location.
 17. The wall should be strong enough to sustain a weight of at least four times of the display and Wall mount ARM/bracket combined. The mounting location must be able to withstand earthquake or other strong shock.
 18. Do not modify any accessories or use broken parts. Contact your dealer with any questions.
 19. Tighten all screws (do not exert excessive force to avoid breaking the screw or damaging its threads).
 20. Drill holes and bolts will be left on the wall once the display and Wall mount ARM/bracket are removed. Stains may be occurred after extended use.
 21. Since the manufacturer has no way to control the wall type and installation of wall mount, the warranty of the product shall only cover the body of the wall mount ARMs. The warranty period of the product is 3 years from the date product manufactured.
 22. Watch out ARM Recoil Hazard ! Please do not remove the cushioning buffer default clamped between two arms when the first time taking it off from the original packing box. It's recommended to remove this designated cushioning buffer when two arms have been well stretched out and ready for install. Handle the ARMs stretched slowly and carefully.
 23. Manufacturer reserves the right to have design changed without prior notice.
 24. Please consult the English language manual for any dispute on conditions.

Various displays have different screw hole specifications. Please check that the Screws (H) fits the specifications for the display to be mounted. Please contact your dealer with any questions.

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